



Pole Syringe Adapter Set

(3ml, 5ml, 10ml sizes – with Needle Keepers)

Included in Package

- 3 × Threaded Pole Syringe Adapters (1 each of 3ml, 5ml and 10ml)
- 15 × Needle Keepers (5 for each syringe size)
- 5 x 3 ml Luer Lock Syringes
- 5 x 5 ml Luer Lock Syringes
- 5 x 10 ml Luer Lock Syringes
- 5 x 3/4" x 16G Needles
- 5 x 1" x 16G Needles

Product Overview

The Garvosa Pole Syringe Adapter allows safe administration of drugs using an extendable pole (such as a Wooster® paint pole or broom handle with metal 3/4" Threads). Ideal for use with exotic wildlife, livestock, or other animals where close contact is unsafe.

The included Needle Keepers are precision-made collars that slide over the needle hub and luer lock syringe tip to keep the needle securely attached during injection—reducing the risk of separation if the animal moves suddenly.

Assembly & Use

1. Prepare the Luer Lock Syringe (**Only Use Luer Lock Syringes**)

- Remove the Plunger from the Syringe
- Remove the Rubber Plunger Seal from the Plunger
- Install the Rubber Plunger Seal on the End of the matching size Pole Syringe Adapter.
- Slide the Pole Syringe Adapter with Seal attached into the Syringe
- Load medicine into the syringe
- Secure the Needle in Place on the Luer Lock Syringe by installing a Needle Keeper

3. Attach Adapter to Pole

- Screw the adapter onto a compatible threaded extension pole.

4. Administer Injection

- Approach animal calmly and steadily.
- Extend pole to the required length and inject medication according to your veterinarian's instructions.
- After Injection carefully remove the Syringe with the Needle and Needle Keeper as one piece and safely dispose of all three pieces as one piece. The Needle Keepers are disposable.

Important Safety Information

- Check local laws before use. Some jurisdictions restrict or regulate remote drug delivery devices.
- Use only under veterinary direction and with legal authorization.
- Wear gloves and protective eyewear when handling medications.
- Keep out of reach of children.
- Dispose of needles safely in a sharps container.

Care & Maintenance

- Wash adapters with warm soapy water after each use.
- Rinse thoroughly and air-dry before storage.
- Inspect Needle Keepers regularly; replace if worn or damaged.

Garvosa Wildlife, LLC

📞 281-732-4781

🌐 www.Garvosa.com

TO REORDER SUPPLIES INCLUDING SYRINGES, NEEDLES and NEEDLE KEEPERS
FOLLOW THE LINK BELOW TO THE GARVOSA OR AMAZON WEBSITE:



3ML Luer Lock
Syringes



5ML Luer Lock
Syringes



10ML Luer Lock
Syringes



16G x 3/4" Needle



16G x 1" Needle



Needle Keepers

Link to the Extension Pole we Recommend Available at your local Home Depot:



Link to www.Garvosa.com to Order Syringe Adapter Sets

